

# Asiga - Max UV 385

## 3D Printing Workflow: Ultracur3D® DM 2505

Technology: DLP

### PRINT SETTINGS

Layer Thickness (µm)	Irradiance (mW/cm <sup>2</sup> )	Exposure Time (s)
100	9	5

	WASH CYCLE 1	WASH CYCLE 2
Agitation Method	Ultrasonic	Ultrasonic
Cleaner	Distilled water	Distilled water
Cleaning Time (min)	4	4



### WAIT BEFORE POST CURING

Waiting Time (min) 30

Temperature (°C) 40

### POST CURING

Model	Otoflash G171
Settings	N.A.
Atmosphere	Ambient
Time (min) or Flashes	3000
Flip & Repeat	Yes

### THERMAL TREATMENT

Ramp up (°C / min)	N.A.
Holding Temp. (°C)	N.A.
Holding Time (h)	N.A.
Ramp down (°C / min)	N.A.